

ASSEMBLY INSTRUCTIONS


Medical Micromanifold Pipe-Away Riser Tube Kits 579 9000 and 579 9100

www.concoa.com


1-800-225-0473

Note: De-pressurize and vent system prior to riser tube installation.


Identify the manifold branch to be converted for use with liquid cylinders:

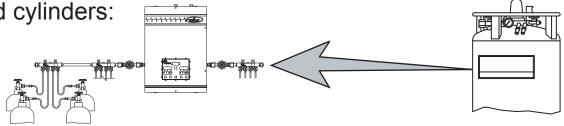


Remove the 1/2" NPT pipe plug on top of the last manifold block on the branch:

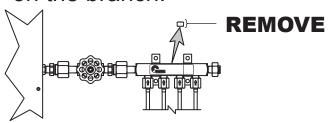


A Check that appropriate inlet pressure appears on tube tag, and apply PTFE tape to NPT thread of riser tube:

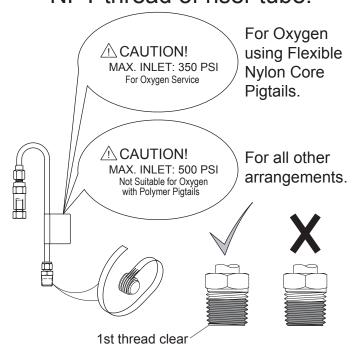

Olean PTFE tape out of the open port taking care to avoid tape falling into the

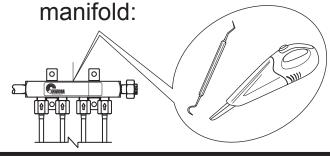

Fasten riser tube to manifold block:

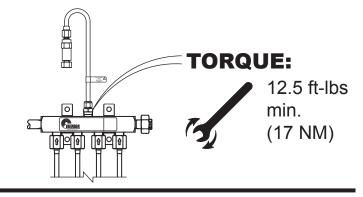

6

Leak test all models regardless of max. inlet shown on the tube tag using 500 psi test pressure.